




QUICK REFERENCE GUIDE

These instructions are supplied as a guide only. Always read the full instruction leaflet before you use LANTUS[®] SoloSTAR[®] for the first time. If you have any questions, ask your healthcare professional.

1 BEFORE YOU START, CHECK YOU HAVE THE RIGHT INSULIN


2 ATTACH A NEW NEEDLE

Keep the needle straight as you attach it.

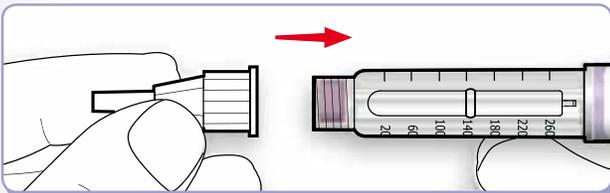


3 PERFORM A SAFETY TEST

This removes air bubbles and ensures that the pen and needle are working properly. Select a dose of 2 units.

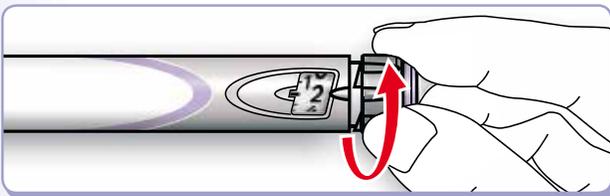

Take off the needle caps. Hold the pen with the needle pointing upwards and press the injection button all the way in. Check if insulin comes out of the needle.

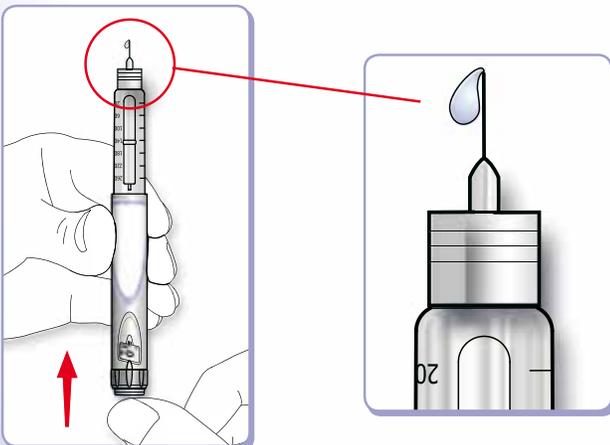


If insulin does not come out, you must repeat the test until it does.

If no insulin comes out after doing the test 3 times, change the needle for a new needle and try again.

4 SELECT YOUR DOSE


(This example shows 30 units.)

The maximum number of units that can be dialed per injection is 80.

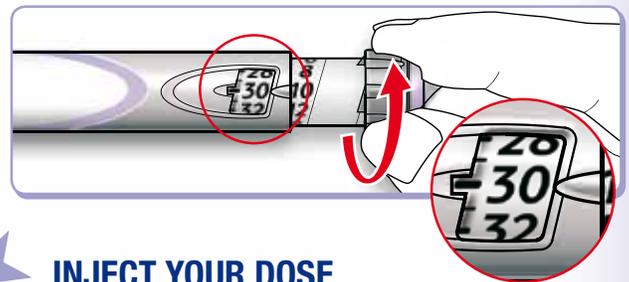

5 INJECT YOUR DOSE

Using the injection method instructed by your healthcare professional, insert the needle into the skin. Press the injection button in all the way. Keep the injection button pressed and slowly count to 10. Then withdraw the needle.

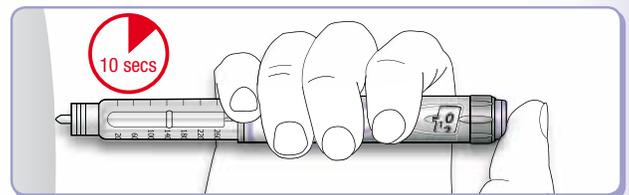

6 REMOVE THE NEEDLE

Put the outer needle cap back on the needle and use it to unscrew the needle from the pen. Dispose of the needle safely, e.g., in a sharps container. Put the cap on the pen.



24 H SUPPORT 1.888.852.6887

IF YOU CAN'T dial to the dose you want, check if you have enough insulin in the reservoir.

IF YOU HAVE any other problems with the pen, first try changing the needle and repeating the safety test.

EACH LANTUS[®] SoloSTAR[®] is for use by one person only.

Copyright © 2008 sanofi-aventis. All rights reserved.
sanofi-aventis Canada Inc., Laval, Quebec H7V 0A3
CDN.GLA.07.08.03E 50086770

